DE EN FR IT

Produktinformationen / Product information
Information sur le produit / Informazioni sul prodotto

siegmund®

FFP3 Atemschutzmaske / Particle filtering half mask Masque filtrant jetable / Facciale filtrante monouso

CE 1463
EN 149:2001+A1:2009 FFP3 NR

Produktname	FFP3 Atemschutzmaske
Product name / Type de produit / Tipo di prodotto	FFP3 Particle filtering half mask / Masque filtrant jetable FFP3 / Facciale filtrante monouso FFP3
Modell	JH-068
Model / Modèle / Modello	
Verordnung	(EU) 2016/425
Regulation / Réglementation / Regolamento	
Material	50% Vliesstoff + 50% Meltblown Vlies
Material / Matériel / Materiali	50% Non-woven fabric + 50% Meltblown fabric / 50% de non-tissé + 50% de tissu Meltblown / 50% Tessuto non tessuto + 50% Tessuto meltblown
Gültigkeitszeitraum	2 Jahre nach Produktionsdatum, siehe innenliegendes Zertifikat
Validity period / Durée de validité / Durata della validità	2 years after production date, see certificate inside / 2 ans après la date de production, voir certificat à l'intérieur / 2 anni dalla data di produzione, vedere il certificato all'interno
Artikelnummer	3377
Article number / Numéro de l'article / Numero dell'articolo	
Prüfnorm	EN 149:2001+A1:2009 FFP3 NR
Test standard / Norme de contrôle / Standard di prova	
PPE Ausstellende Behörde	POLSKI REJESTR STATKOW S.A.
PPE issuing authority / Autorité de délivrance DPI / Autorità emittente DPI	
Adresse der ausstellenden Behörde	Al. Gen. Jozefa Hallera 126 80-416 GDANSK - Poland
Address of issuing authority / Adresse de l'autorité émettrice / Indirizzo dell'autorità emittente	
Notifizierte Stelle	NB1463
Notified Body / Organisme d'inspection notifié / Organismo di controllo notificato	
Import in die EU & Vertrieb	Siegmund Care GmbH
Import to the EU & Sales / Importateur et distributeur dans l'UE / Importatore & Distributore nelle EU	Landsberger Straße 180, 86507 Oberottmarshausen, Deutschland / Germany / Allemagne / Germania, Telefon: (+49) 0 82 33 / 84 89 19 - 0, Internet: www.siegmund.care
Hersteller	Ningbo Haishu Junheng Fashion Co., Ltd.
Manufacturer / Fabricant / Fabbicante	Fengjia Village shiqi Haishu Areh Ningbo Zhejiang, China

DE EN FR IT

Produktinformationen / Product information Information sur le produit / Informazioni sul prodotto

siegmund®

DE Warnhinweise

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EN Warnings

1. Before use, the wearer must read and understand these instructions. Please save these instructions for reference.
2. This product is a disposable product. It is strictly prohibited to reuse it. It should be treated according to the requirements of the hospital and environmental protection department after use.
3. Before use, please check whether the package is in good condition and the package or mask is damaged. Do not use it. The external packaging mark, production date and validity period shall be confirmed and used within the validity period.
4. It should be used with caution for those allergic to non-woven fabrics.
5. In the process of use, if the respiratory resistance is significantly enhanced (when breathing is difficult), the mask is damaged, and the mask is contaminated (with foreign matters such as blood stains and droplets), it should be replaced in time.
6. The respirator does not produce oxygen. Please do not use in the air with oxygen concentration lower than 19.5%.
7. When the concentration of the pollutant reaches an immediate life-threatening and health-threatening concentration, or the concentration is unknown, or when the concentration exceeds 10 times the allowable exposure limit (PEL) or the concentration specified by applicable government regulations, the lower one shall prevail. Please do not use this respirator.
8. Do not use when whiskers or other facial hair or conditions interfere with the good fit of the face and the sealing surface of the respirator.
9. For people with certain respiratory diseases, such as asthma or emphysema, consult a physician and complete a medical assessment before using the respirator.
10. The user of this respirator should be an adult who has been trained in proper use methods and understands the use restrictions. Protective masks are not designed for use by children.
11. Storage is recommended in a dry, ventilated, non-corrosive air environment, away from fire and flammable materials.

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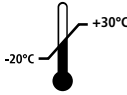
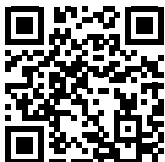
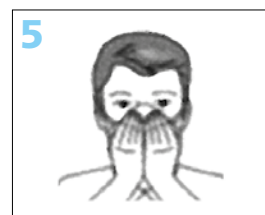
EN Instruction Manual

1. Cup the respirator in your hand with the nosepiece at your fingertips allowing the headbands to hang freely below your hand.
2. Position the respirator under your chin with the nosepiece up.
3. Pull the top strap over your head resting it high at the top back of your head. Pull the bottom strap over your head and position it around the neck below the ears.
4. Place the fingertips of both hands at the top of the metal nosepiece. Mold the nosepiece to the shape of your nose by pushing inward while moving your fingertips down to both sides of the nosepiece. Pinching the nosepiece using one hand may result in less effective respirator performance.
5. The seal of the respirator on the face should be fit-checked prior to wearing in the work area.
 - a) Cover the front of the respirator with both hands. Being careful not to disturb the position of respirator.
 - b) Inhale sharply. A negative pressure should be felt inside the respirator. If any leakage is detected, adjust position of respirator and/or tension of strap. Retest the seal. Repeat the procedure until the respirator is sealed properly.

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